

Impression TRANSFER in THREE SECTIONS For PICK-UP impression technique

Code **01TR333** - Code **01TR403**

The transfer in three sections has been studied to **solve** the **disparallelism** problems with the internal connection implants.

Contrary to all the other transfer types actually on the market, this transfer **allows to obtain a precise impression** even in case of **strong disparallelism between implants**.


This transfer is composed of 3 sections:

1. Body,
2. Removable cylinder with hex,
3. Passing screw.

As the **hex section is removable**, once the impression has been taken, you can remove the passing screws and the cylinders with the hex, before removing the impression tray.

With this system, as the hexes have been removed and the disparallelisms eliminated, the extraction of the impression tray can be achieved without tensions and without affecting the precise positioning of the transfers in the impression material

The transfer in three sections are available for internal hex implants (see table).

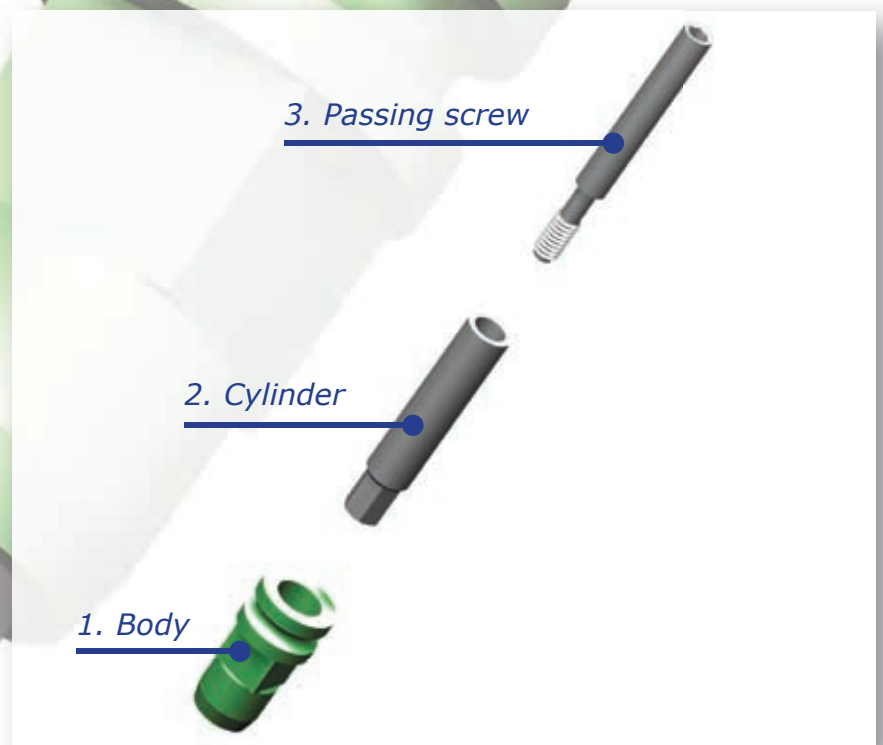

Colour	Code	For implants
Green	01TR333	Platform ø 3,5mm
Yellow	01TR403	Platform ø 4,0m

LEADER
I T A L I A